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Mounting and Instructions Manual for ErgoPedal, ErgoPedal with Safety Release, ErgoPedal with Calf Brace

Fig. 1: ErgoPedal with Safety Release

Fig. 2: ErgoPedal with Calf Brace

Description and Intended Use

An ErgoPedal positions and fixes the foot, allowing the leg to be actively or passively supported and mobilised when riding a tri-cycle. It can help compensate for functional impairments. One-handed Velcro fasteners with spring steel inserts make it easy to step into the pedals.

These instructions refer to 3 models: The **ErgoPedal** is screwed onto a standard pedal. The **ErgoPedal with Safety Release** is mounted on a SHIMANO-SPD-Pedal. It offers joint-friendly, lateral freedom of movement when pedalling at an angle of approx. +/- 3° and a safety release function in critical situations. The **ErgoPedal with calf brace** is screwed onto a standard pedal and additionally provides support and fixation for the lower leg.

Each version is available as a right or left ErgoPedal and in 2 sizes (small 20 - 28 cm, normal 27 - 34 cm) depending on shoe length.

All suspected serious incidents occurring in connection with the ErgoPedals should be reported to your physician or HP VELOTECHNIK. In addition, you can also contact the responsible authority directly. Suspected serious incidents are incidents resulting in death, worsening of health condition or serious danger to public health.

Danger! With single-track recumbent bicycles, it must be possible to place the feet quickly and securely on the ground when stopping in order to prevent a fall. This is possible only to a limited extent or not at all with ErgoPedals. Do not use any version of the ErgoPedals with single-track recumbents or bicycles.

Assembly

All work on the pedals is safety related. Retrofitting must therefore be carried out by a qualified cycle mechanic.

Mounting the Lower Leg Support

Remove the insertion rod with calf pad from the packaging. Loosen the 2 clamps (Fig. 5, a), turn the holding rod upwards. Push the insertion rod from above into the holding rod.

Mounting of the ErgoPedal

Lubricate the pedal thread with assembly paste or grease. Depending on the mounting side, note the direction of rotation of the pedal thread in the crank: the left crank has a left-hand thread, the pedal is tightened anti-clockwise. The right crank has a right-hand thread and the pedal is tightened clockwise. Use a 6 mm Allen wrench or a 15 mm open-end wrench to tighten the pedal axle into the thread of the pedal crank arm to a torque of 35 - 55 Nm.

Adjustment options of the Retaining Straps (all Models)

On the "small" models, both straps are the same length. On the other models (no size specified in the name), the retaining straps are different lengths. The longer one is intended for the

Attention! HP VELOTECHNIK ErgoPedal models are medical products and may be used for therapeutic purposes and are suitable for rehabilitation measures. If you wish to use them for this purpose, please consult your doctor or therapist before use and discuss the effects on your health!

Safety Instructions

Having received the product, check it for completeness. If the product is damaged, contact your retailer immediately and do not use the product.

Reuse

HP VELOTECHNIK ErgoPedals are suitable for reuse if used as intended and after inspection by a specialized dealer.

Product lifetime

Taking into account the operating instructions, frequency of use, regular and correct maintenance and correct use, the expected service life is 5 years.

Regulation of the European Union

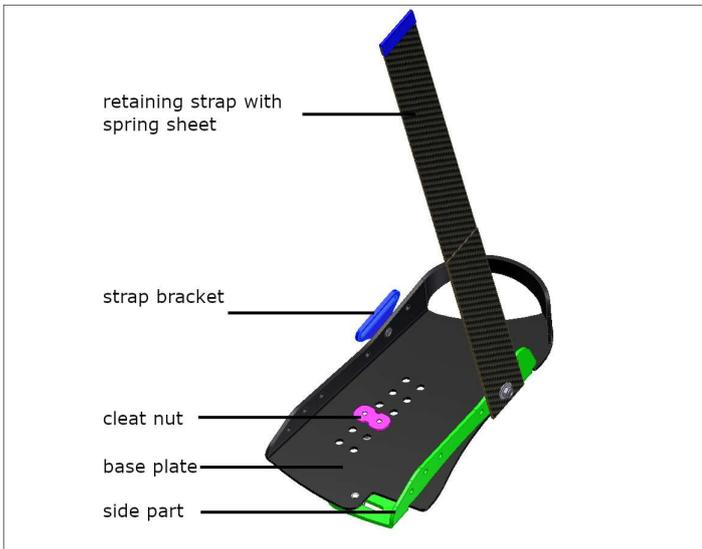
Fig. 3.: Part designation using the example of an ErgoPedal with Safety Release

rear position.

You can use 1 or 2 straps and adjust their position using the holes in the side plates of the ErgoPedal.

Remove the lock nuts and remove the retaining strap and strap bracket. Use a light hammer blow on the press-fit threaded bolts to loosen them from the holes.

Attach the strap bracket to the inside of the ErgoPedal and the retaining strap to the side bracket in the position that suits you best. To do this, insert the bolt into the hole and prevent it from falling out while you fit and tighten the guide bracket / strap, washer and self-locking nut. The threaded bolt is self-locking. Ensure that the end of the threaded bolt is not pulled under the surface of the side bracket.

If the threaded bolt is carried by the self-locking nut and turns, the threaded bolt can first be pulled into the hole with an M5 nut without locking until it is secured against turning. This must then be replaced with a self-locking nut.

Adjusting ErgoPedals with Safety Release

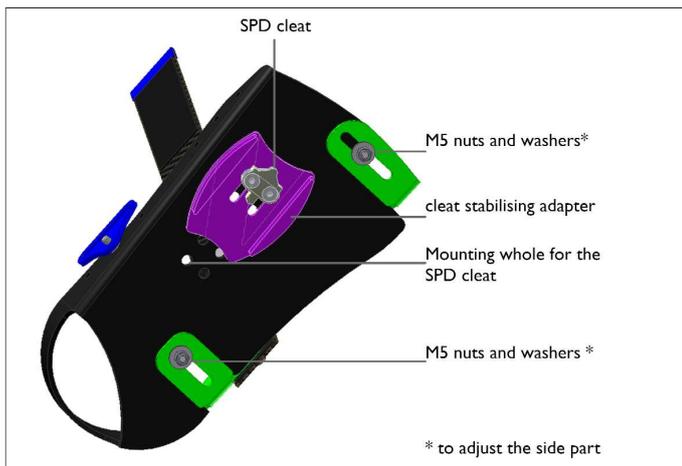
Fig. 4: ErgoPedal with Safety Release from below

The following describes the adjustment possibilities of the The adjustment options for the ErgoPedal with Safety Binding on the SHIMANO pedal are described below. For the adjustment options of the parts of the SHIMANO SPD system, refer to the SHIMANO Owner's Manual. If this is not included, you can find it on the Internet by looking up the part number on the SHIMANO SPD pedal.

1. Foot Position over the Pedal Axle

Loosen the ErgoPedal attachment from the SPD pedal by turning it sideways. Remove the 2 countersunk screws from the SPD cleat. Remove the cleat with double washer, cleat protector and cleat nut from the other side of the base plate. Insert the cleat nut into another pair of holes and refit parts (tightening torque 5 – 6 Nm).

2. Angle of the Foot over the Pedal Axle.

To adjust the angle of the pedal over the pedal axle, loosen the 2 countersunk screws in the SPD cleat. Rotate the cleat and cleat protector to the desired position and retighten the countersunk screws (tightening torque 5 – 6 Nm).

3. Adjustment to the shoe width

Loosen the 2 nuts on the bottom of the ErgoPedal. Slide the side plate into the appropriate position and retighten the nuts (fig. 4, tightening torque 5 – 6 Nm).

4. Pedal Distance

The cleat can be moved sideways by loosening its fixing screws. Follow the instructions of the SHIMANO SPD system (tightening torque 5 – 6 Nm).

Adjusting ErgoPedal and ErgoPedal with Lower Leg Fixation

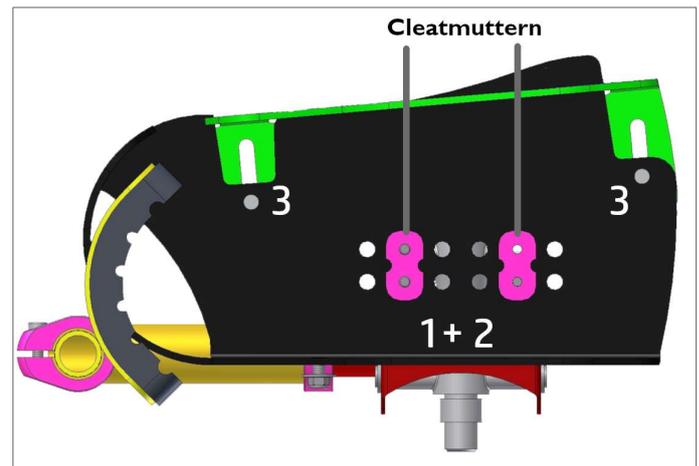
Fig. 5: Elements for adjusting the foot position on the ErgoPedal and ErgoPedal with lower leg fixation, seen from above. (The picture shows an ErgoPedal with lower leg fixation).

1 + 2: Position and angle of the foot above the pedal axle
3: adaptation to the width of the foot

1. Foot Position above the Pedal Axle

The base plate is attached to the pedal by a stainless steel cage. Remove the 4 cylinder head screws from the base plate and lift off the 2 cleat nuts. Slide the pedal forward or backward along the row of holes to the position that you prefer. Mount the pedal with the cleat nuts on the top of the base plate and the 4 cylinder head screws and washers on the bottom of the base plate (tightening torque 5 – 6 Nm).

2. Angle of the Foot above the Pedal Axle

To adjust the angle of the pedal over the pedal axle, loosen the 4 cylinder head screws that secure the pedal under the base plate. Rotate the pedal to the desired position and retighten the cylinder head screws (tightening torque 5 – 6 Nm).

3. Adaptation to the Shoe Width

Loosen the 2 nuts of the side part (fig. 5; number 3). Slide the

side part into the correct position and retighten the nuts.

4. Pedal Distance

Loosen the 4 cylinder head screws at the front and rear of the stainless steel cage and move it sideways on the pedal body. If this adjustment is not sufficient, the base plate can be moved on the stainless steel cage or secured in another hole in the stainless steel cage (tightening torque 5 – 6 Nm, not shown).

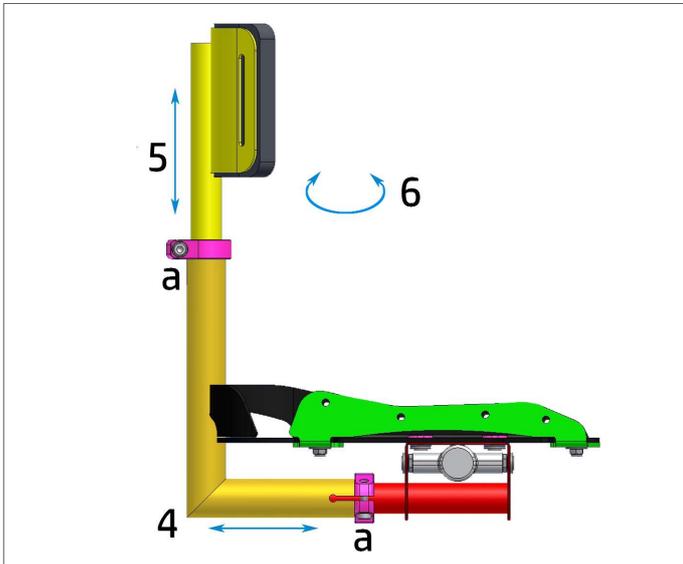
Fig. 6: Adjusting the lower leg fixation
a clamps;
4 distance of the calf pad;
5 height adjustment of the calf pad;
6 rotating the insertion rod with calf pad

The holding and insertion rods can be pulled out, pushed in or rotated by loosening the clamps, as shown in Figure 6. Once you have adjusted the lower leg fixation to your leg, tighten the screws of the clamps (tightening torque 5 – 6 Nm).

Final Check

Once the ErgoPedal has been fitted to the crank and adjusted to you, check its function. When pedalling, the pedal must not touch the crank arm or any other parts of the trike.

Using des ErgoPedals

To get into the ErgoPedal, first take a seat on the trike. Open the retaining strap(s) by pulling on the blue flag and loosening the Velcro fastener. The strap should not be pulled out of the strap bracket. The integrated spring sheets lift the strap(s) so that the foot can easily be guided in.

Now place your foot on the ErgoPedal, tighten the strap(s) and place the Velcro partners on top of each other.

To secure the lower leg, open the Velcro strap on the calf pad, place it around the leg and Velcro it back onto the rear part of the calf support.

Fig. 7: Positioning of the Velcro strip to hold the lower leg fixation in place

Pushing a trike with ErgoPedal with lower leg fixation:

Mount the Velcro strip on top of the main frame, above the quick release levers (fig. 7).

When the trike is parked, turn the crank to the lowest position and attach the retaining strap of the lower leg fixation so that the holding rod hangs approximately horizontal to the ground. You can now easily push the bike forward.

It is now also possible to push the bike backwards. The cranks can turn without parts of the ErgoPedal coming into contact with the ground.

However, push the bike backwards carefully to avoid collisions between the ErgoPedal and parts of the frame or the wheel suspension.

Reflector set for the ErgoPedal (all models, available as accessories)

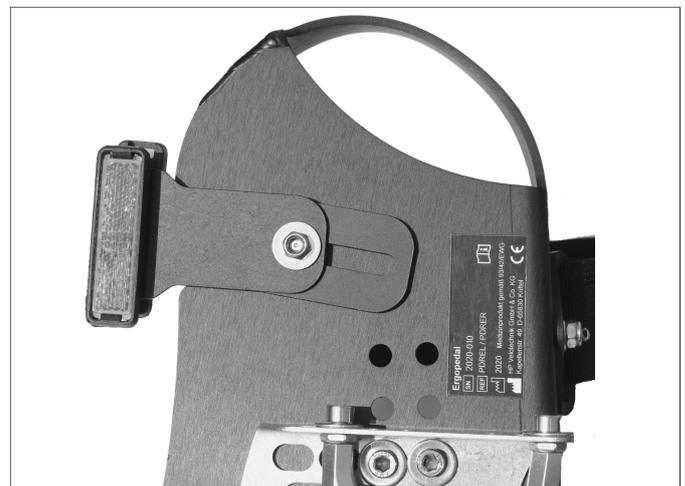
Fig. 8: ErgoPedal with mounted reflector

Mount the reflector bracket to the underside of the base plate. To do this, use the mounting point of the side part in the heel area (fig. 8).

⚠ Attention! When parking or pushing backwards a trike with a mounted ErgoPedal with lower leg fixation, the pedals rotate and the lower leg fixation can drag on the ground or contact other trike parts. This can cause damage. Mount the enclosed Velcro holder with which the lower leg fixation is lifted.

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Labelling of the ErgoPedal

On the underside of the ErgoPedals there is a marking with symbols according to DIN EN ISO 15223-1.

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