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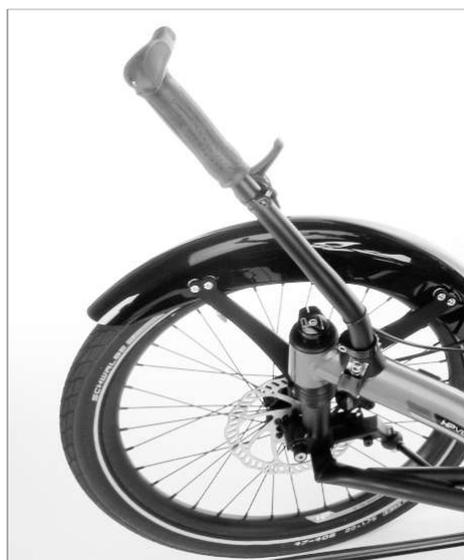
Mounting Instruction HP VELOTECHNIK Standing-up Aids for Scorpion Models with Front Suspension

Standing-up aid mounted on a Scorpion plus 20

Description

HP VELOTECHNIK standing-up aids are handles that can be mounted left, right or in pairs above the main frame's cross tube. The standing-up aids provide a firm hold to pull yourself up from the seat or to support yourself when getting in. The standing-up aids can be adjusted or removed after loosening a quick release. Mounting can be performed by customers.

Content

- standing-up aid with quick release lever adapter
 - hose clamp
 - mounting instructions
 - mounting material
- (If necessary, pull the adapter out of the standing-up aid to remove the mounting material).

Mounting

Position the adapter onto the cross tube in such a way that the mounting tube points slightly outwards.

Place the hose clamp around the cross tube.

To fix the adapter, first use the longer cylinder head screw (45 mm) to preform the hose clamp. Then replace the screw with the shorter cylinder head screw (40mm), placing a washer under the screw head. Tighten the screw with a torque of 8–10 Nm.

Put the standing-up aid on the adapter, align it and fix the position by closing the quick release lever. Find further information on quick release levers in your trike's manual.

Check if the handlebars can be moved completely in both directions. The clearance between the rider's hands and the standing-up aid must be at least 2 cm in any position of the handlebars.